



MolecuLight i:X™ QUICK START GUIDE


1. CLEANING

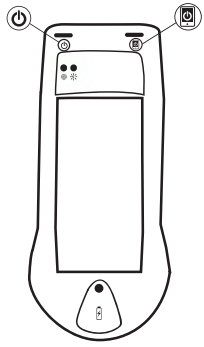


a. Ensure *i:X* is OFF or turn device OFF and Display Screen OFF.

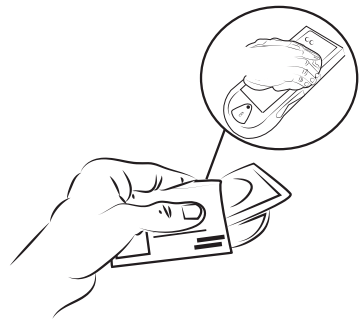


b. Use a disinfecting wipe to preclean the *i:X*.

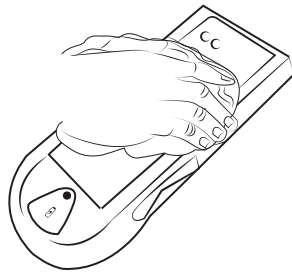

c. Use a second disinfecting wipe to clean the whole device including all crevices.

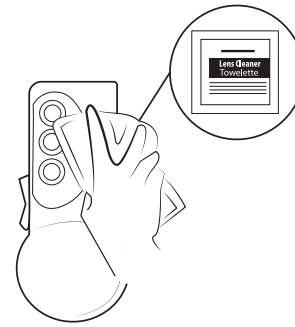

d. Use lens towelette to clean all glass surfaces.

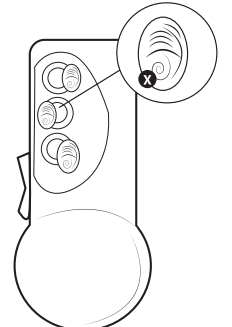


e. Check that all windows are clean, free of all debris and fingerprints.


2. CHARGING




a. Charge device and Display Screen with the MolecuLight Power Cable.

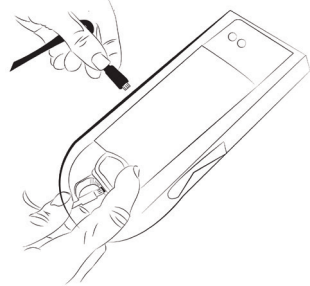


b. One end of the Power Cable connects to the device and the other to the Display Screen.

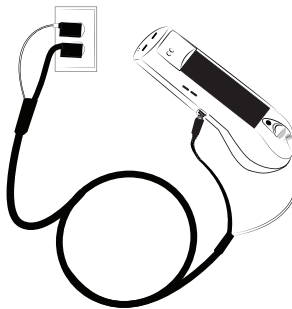

c. Both charging ends should be plugged into wall connector.

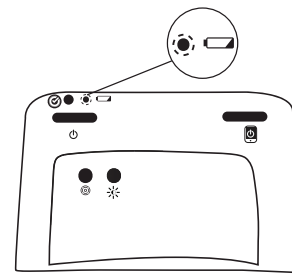


d. LED flashes when device is being charged.

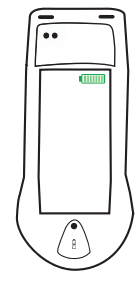


e. Battery icon will be visible when Display Screen is being charged.


3. TURNING ON

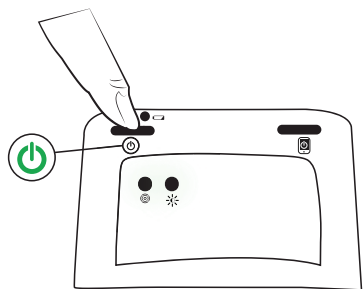


a. Push power button (3 sec) to turn device on.

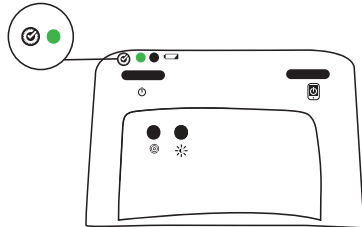


b. Check that the System Status LED will be green (if yellow or red consult User Manual).

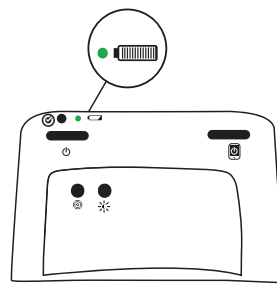

c. Verify the battery status on device (green indicates maximum charge capacity),

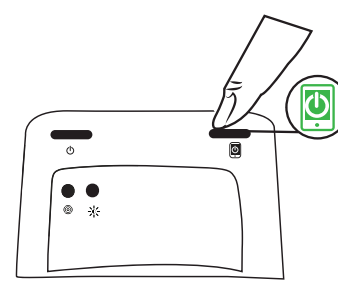


d. Push screen power button (3 sec) to turn Display Screen on.

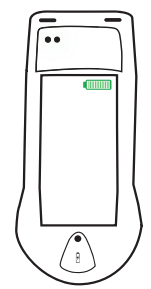


e. Verify Display Screen battery capacity.


4. CAPTURING A STANDARD LIGHT IMAGE/VIDEO

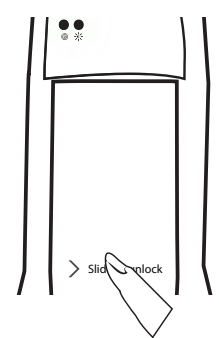


a. Press the home button and select slider.

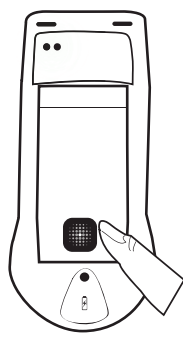


b. Select the iX Camera App on Display Screen.

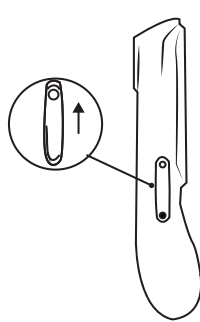


c. Position Rocker Switch in Up position.

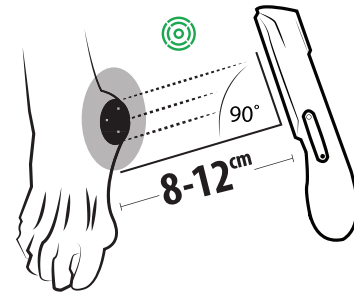

d. Use Range Finder Sensor LED to reach appropriate distance (LED turns green).

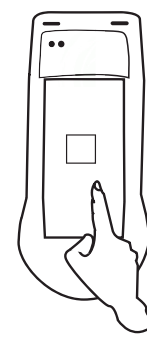


e. Tap on the screen to improve focus as required.

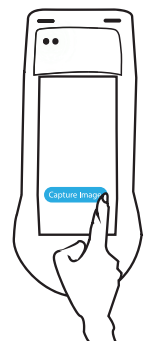


f. Capture image and/or video, Range Finder Sensor LED should remain green.


5. CAPTURING A FLUORESCENCE LIGHT IMAGE/VIDEO

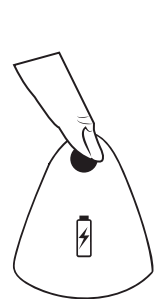


a. Use home button to access screen, select slider if needed.

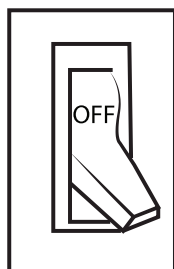


b. Turn off room lights.

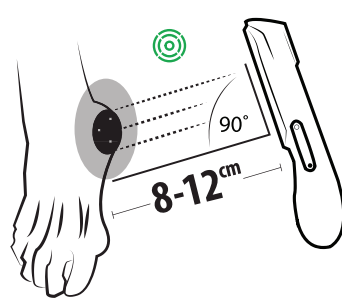


c. Use Range Finder Sensor LED to reach appropriate distance (LED turns green).

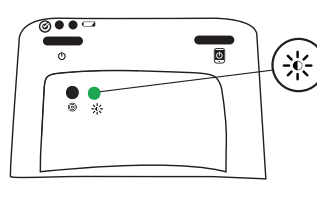


d. Confirm Ambient Light Sensor LED is green before toggling Rocker Switch to FL-Mode.

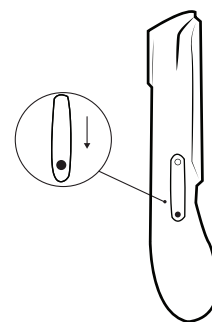

e. Toggle to FL-Mode; Ambient Light Sensor LED turns off when FL-Mode is activated.


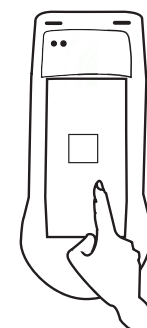


f. Tap on the screen to improve focus if required.

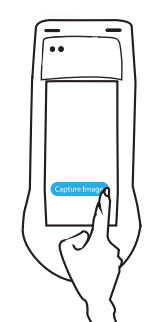


g. Capture image and/or video, Range Finder Sensor LED should remain green.

6. TURNING OFF

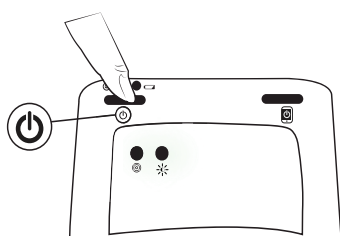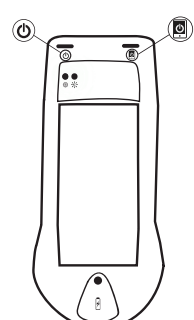

a. Turn device off by pressing power button for 3 seconds.

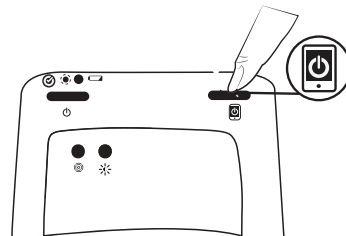

b. Turn Display Screen off by pressing screen power button for 3 seconds.



MolecuLight

©2017 MolecuLight Inc. All Rights Reserved. PN 1295 Rev 1.0.
Consult the MolecuLight i:X™ User Manual for more information.